



**How many fall risk patients are you caring for?  
How often can you check on them?**

**WE CAN HELP!**

## Basic-Check™

- **Monitor chair, toilet, wheelchair, or bed.**  
Uses any Bed-Check® Sensormat®.
- **No on-off switch to remember. Monitors automatically.**  
Helps you meet JCAHO requirements.
- **Total portability.**  
Common, easily replaced 9 volt battery.
- **No settings or adjustments.**  
Just a reset button that silences the alarm and puts the system on hold until weight is again sensed on the Sensormat.
- **Low battery charge alert** (a low continuous tone).
- Meets the same **UL® and CSA Standards** as our other monitors.
- Engineered to withstand extreme environmental static discharge.
- **100% tested** before shipment.
- **Rugged design with integral hanger and Velcro®.**

*Weight on Sensormat, Basic-Check monitors.  
Weight off Sensormat, Basic-Check alarms.  
Push button to hold until weight is back  
on Sensormat.*

**Simple. Easy. Basic.**  
*Basic-Check from Bed-Check.*



**In a Chair**



*Chair  
Sensormat*



**On a Toilet**



*Potty-Check®  
Sensormat*



**In a Bed**



*Bed Sensormat*





# Basic-Check™

## Fall Risk Monitoring Control Unit

**Purpose:** The Basic-Check Control Unit is intended to be used in conjunction with a Bed-Check Sensor-mat® as a system for the monitoring of fall risk patients or residents in a bed, chair, wheelchair, or on a toilet. Basic-Check provides an audible alarm to alert the caregiver when the person begins to get up unassisted.

**Components:** The System consists of the Basic-Check Control Unit, and a pressure sensitive Sensormat (ordered separately) with a cord that connects to the Basic-Check. Sensormats are available for bed, chair/wheelchair, and toilet. The Basic-Check has a convenient integral hanger, and a Velcro® strip.

**Power requirements:** Basic-Check is powered by a common nine volt alkaline battery. Expected battery life is normally over a year unless alarming is very frequent.

**Installation:** The Basic-Check is hung on the bed headboard or frame, the chair/wheelchair back, or wherever convenient adjacent to the toilet. It should be inaccessible to the monitored person. The appropriate Sensormat is installed per its Technical Bulletin. See the Basic-Check User Guide for complete instructions.

**In-Service:** As with all Bed-Check Corporation products, complete personal in-servicing is available from your professional Bed-Check Distributor.

### Features and Operation:

The Basic-Check is the easiest and simplest of all Bed-Check's Control Units to operate.

- Monitoring begins immediately when patient weight is sensed on the Sensormat.
- If weight is then removed from the Sensormat, the Basic-Check immediately sounds an audible alarm tone.
- Weight back on the Sensormat silences the alarm, and monitoring resumes.
- Pushing the reset button silences the alarm and puts the system on hold until weight is again sensed on the Sensormat.
- A low continuous tone warns you when the battery charge is low.

### Options:

The Basic-Check is designed to be a simple and economical Control Unit, and no options are offered. If there are needs not met by the Basic-Check, there are many options and accessories available with other Bed-Check Control Units.

*The Basic-Check is Part # 72000*