

CHAIR-CHECK® II IN-SERVICE REVIEW (WHEN USED WITH BED SENSORMAT®) for BED Service

(consult Chair-Check® II User Guide for more detail)

PROPER INSTALLATION OF THE CHAIR-CHECK II SYSTEM AND BED SENSORMAT



A. Attach a rubber band and clip to each end of the Sensormat®. This important step will keep the Sensormat positioned correctly on the mattress to ensure reliability. The Sensormat will work with either side facing up.



B. Make sure all sheets and covers are pulled down to the foot of the bed. Place Sensormat 3 to 5 inches below the bend in the mattress.



C. Attach the alligator clips to the fabric of the mattress, mattress piping or bed springs on both sides. You may now reposition sheets and covers.

Record date put in use on Sensormat cord tag.



D. Plug the Sensormat's telephone-style jack into the Chair-Check II control unit.

IMPORTANT: Use only Bed-Check Sensormats with the Chair-Check II to avoid known reliability problems associated with inferior mats sold as "equivalent."



E. If a rechargeable nickel-metal hydride battery is installed, the power supply/charger may be plugged in if to be used.



F. To secure the unit out of reach of the patient/resident, attach the Chair-Check II control unit to the bed frame* or headboard using the standard metal clip or an optional wire holster.

**Not directly under the mattress fold, to avoid urine damage.*



G. Help patient/resident into bed. The control unit will arm itself automatically. The alarm will beep one time, followed by a blinking red light located next to the RESET button.



H. When the patient/resident attempts to exit the bed, an alarm will sound in the room. Press the Reset button once to cancel alarm. Unit will arm itself again automatically if the patient returns to bed.

IMPORTANT:

Check for System Effectiveness Daily and/or at shift change.

Always check correct positioning of Sensormat after any alarm.

Use Bed-Check products only to maintain Bed-Check Warranty and to ensure System compatibility.

The physical characteristics, habits and factors that make each patient at risk for falls are different and may change daily. Please reassess as necessary.

The Chair-Check II provides an audible alarm only. For use with nurse call system, consult Bed-Check for other systems.



1-800-523-7956 • www.bedcheck.com
P.O. Box 170 • Tulsa, OK 74101

©2002 Bed-Check Corporation. All rights reserved.