

CHAIR-CHECK® II IN-SERVICE REVIEW

for BED

PROPER INSTALLATION OF THE BED-CHECK® CHAIR-CHECK II SYSTEM ON A BED

For more information, consult Chair-Check II User Guide



A. Remove Sensormat® from plastic bag. Attach a rubber band and clip to each end of the Sensormat. This important step will keep the Sensormat positioned correctly on the mattress to ensure reliability. The Sensormat will work with either side facing up.



B. Make sure all sheets and covers are pulled down to the foot of the bed. Place Sensormat 3 to 5 inches below the bend in the mattress.



C. Attach the alligator clips to the fabric of the mattress, mattress piping or bed springs on both sides. You may now reposition sheets and covers. Record date Sensormat put in use on cord tag.



D. Plug the Sensormat phone-style jack into the Chair-Check II control unit.

IMPORTANT: Use only Bed-Check Sensormats with the Chair-Check II to avoid known reliability problems associated with inferior mats sold as "equivalent".



E. If a rechargeable nickel-metal hydride battery is installed, the power supply/charger may be plugged in to be used.



F. To secure the unit out of reach of the patient/resident, attach the Chair-Check II control unit to the bed frame* or the headboard, using the wire hanger or the Velcro® backing.

*Not directly under the mattress fold, to avoid urine damage.



G. Help patient/resident into bed. The control unit will arm itself automatically. The alarm will beep one time, followed by a blinking red light located next to the Reset button.



H. When the patient/resident attempts to exit the bed, an alarm will sound in the room. Press the Reset button once to cancel alarm. Unit will monitor again automatically if patient returns to bed.

To remove the patient from the bed, press the Reset button and the Chair-Check II will go into stand-by mode. If the patient has not been removed from the bed in one minute, the unit will resume monitoring. This will cause an audible beep and the red blinking light will resume.

IMPORTANT:

Check for System Effectiveness Daily and/or at shift change.

Always check correct positioning of Sensormat after any alarm.

Use Bed-Check products only to maintain Bed-Check warranty and to ensure system compatibility.

The physical characteristics, habits and factors that make each patient at risk for falls are different and may change daily. Please reassess as necessary.

The Chair-Check II provides an audible alarm only. For use with nurse call system, consult Bed-Check for other systems.

STANLEY



P.O. Box 80238, Lincoln, NE 68501

1-800-523-7956 • www.bedcheck.com