



**Some of these folks are fall risks.
Chair-Check® II helps me know if they try to get up unassisted.**

Chair-Check® II

- **Monitors chair, toilet, wheelchair, or bed.**
Uses any Bed-Check® Sensormat®
- **Total portability**
Common 9V battery, with very long life, and low charge alert. Integral battery containments. No door to fall off - no duct tape required! Optional rechargeable battery and charger/AC adaptor.
- **No on-off switch to remember. Monitors automatically.**
Helps you meet JCAHO requirements.
- **Automatic monitoring restart**
Silences and resumes monitoring with weight back on the Sensormat. Fewer nuisance alarms.
- **Selectable alarm tones, volume, and delay**
3 tones, 2 volumes, 0 to 5 second delay. Customize to the individual.
- **Conforms to UL® and CSA Standards**
UL 1069 and CSA C22.2 No. 205R, as tested by ETL.
- **Rugged durability**
Test conformance requires test dropping 4 feet onto concrete 100 times and still function! Engineered to withstand extreme environmental static discharge (>4000 volts).
- **100% tested**
Every monitor is functionally tested before shipment. Two year warranty.
- **Read the ECRI Report**
ECRI's Health Devices, September 2004, for ratings of this and competitive products.



On a Toilet



In a Chair



In a Bed



In a Wheelchair

Rugged design and ability to monitor almost any seated or reclined application make the Chair-Check II a universally adaptable fall risk monitoring system. If you must choose just one system for bed, chair, and toilet use, choose the Chair-Check II.



30 or 90 Day Bed Disposable Sensormats

Chair Disposable Sensormat



Potty-Check® Disposable Sensormat



Chair-Check® II

Fall Risk Monitoring Control Unit

Purpose: The Chair-Check II Control Unit is intended to be used in conjunction with a Bed-Check Sensormat® as a system for the monitoring of fall risk patients or residents in a bed, chair, wheelchair, or on a toilet. Chair-Check II provides an audible alarm to alert the caregiver when the person attempts to get up unassisted.

Components of the System: The Chair-Check II Control Unit, and a pressure sensitive Sensormat (ordered separately) with a cord that connects to the Control Unit. Sensormats are available for bed, chair/wheelchair, and toilet. Chair-Check II has a convenient integral mounting clip.

Power requirements: Chair-Check II is powered by a common nine volt alkaline battery. See Options for more data.

Installation: The Chair-Check II is hung on the bed headboard or frame, the chair/wheelchair back, or wherever convenient adjacent to the toilet. It should be inaccessible to the monitored person. The appropriate Sensormat is installed per its in-service instructions. See the Chair-Check II User Guide for complete instructions.

In-Service: As with all Bed-Check products, complete personal in-servicing is available from your professional Bed-Check Distributor.

Features and Operation: The Chair-Check II is the most versatile Control Unit in Bed-Check's product line.

- Battery or AC operation (with rechargeable battery and AC Adaptor) allows use in portable or bed applications.
- Common 9V alkaline battery. Battery life is "the gold standard" in the business, with low charge alert. Secured battery access.

- Controls for hold, tone, and delay are designed for easy changes by the caregiver, but not the patient.
- Caregiver can put the System on Hold while servicing the patient. Monitoring begins again automatically when the patient's weight is back on the Sensormat.
- Multiple tones allow Chair-Check II to be distinguished from other equipment or other Chair-Checks.
- Delay is 0 seconds for most seated applications, since the patient is postured for quick stand up. 1, 3, or 5 seconds may be set for other circumstances, such as bed monitoring.
- Utilizes low cost, disposable, pressure sensitive Sensormats available for chair, bed or toilet monitoring. (Not intended for use with other manufacturers' mats.)
- Non-latching alarm action. The alarm ceases and monitoring automatically resumes when weight is again sensed. Reduced "nuisance alarms".
- Superior mechanical and electronic engineering for the best durability in the business.
- Integral stainless steel clip. (Also see Options.)
- Features covered by numerous patents.

For the Chair-Check II, order *Part # 72010*

Options:

- Nickel-metal hydride rechargeable battery allows recharging >1000 times.
- AC Adaptor/battery charger. (Requires rechargeable battery.) Allows system use while charging the battery.
- Holsters – Wire basket holster for easier mounting on chairs and headboards. With 1" or 2" depth hanger.