

CLASSIC-CHECK™ IN-SERVICE REVIEW

PROPER INSTALLATION OF THE CLASSIC-CHECK SYSTEM ON A BED WHEN USED WITH A BED SENSORMAT®

(Also consult the Classic-Check User Guide for setting changes.)



A. Remove the Sensormat from the plastic bag, and attach a rubber band and clip to each end. This important step will keep the Sensormat positioned correctly on the mattress to ensure reliability. The Sensormat will work with either side facing up.



B. Make sure all sheets and covers are pulled down to the foot of the bed. Place the Sensormat 3 to 5 inches below the bend in the mattress.



C. Attach the alligator clips to the fabric of the mattress, mattress piping or bed springs on both sides. You may now reposition sheets and covers. Record date put in use on Sensormat cord tag.



D. Plug the Sensormat into the Classic-Check Control Unit.

IMPORTANT: Use only Bed-Check Sensormats with the Classic-Check control unit.



E. Plug in the Classic-Check power supply.

F. If connecting to the nurse call system (not required), plug the nurse call cord into the facility's nurse call interface.

Note: An adaptor may be required.



G. To secure the unit out of reach of the patient/resident, attach the Classic-Check control unit to the bed frame using the standard Velcro® strap, or to the headboard or wall using an optional wire holster.



H. Help the patient/resident into bed. The Classic-Check will arm itself automatically. The yellow hold light will count down 5 seconds, and then beep once as the green MONITOR light illuminates. You can bypass the 5 seconds by pressing the Reset button once.



I. When the patient/resident attempts to exit the bed, an alarm will sound in the room and at the nurse station if connected in F above. Press the Reset button once to cancel alarm. Be sure to reset the nurse call station in the patient's/ resident's room.

J. To remove the patient from the bed, press the Reset button. The unit will enter the temporary HOLD mode for 25 seconds (the yellow light will blink). At 25 seconds, with no weight on the Sensormat, the unit will begin permanent HOLD (steady yellow light).



K. If at 25 seconds the patient is still on the Sensormat, Classic-Check will begin monitoring again (green light). **INSTANT HOLD OR MONITOR:** At any time there is weight on the Sensormat, you may instantly go from MONITOR to HOLD, or from HOLD to MONITOR by pressing Reset. **Always look for the green MONITOR light before leaving the patient/resident.**

IMPORTANT:

Check for System Effectiveness Daily and/or at shift change.

Always check correct positioning of Sensormat after any alarm.

Use only Bed-Check products to maintain Bed-Check Warranty and to ensure System compatibility.

The physical characteristics, habits and factors that make each patient at risk for falls are different and may change daily. Please reassess as necessary.



Bed-Check
CORPORATION
Reducing Falls Through Innovative
Technology Since 1978

P.O. Box 170, Tulsa, Oklahoma 74101
1-800-523-7956 • 1-918-592-3338
Fax: 1-918-582-9828
www.bedcheck.com