

5. While Chair-Check II is monitoring, the red light will flash at intervals.

6. To remove the patient from the chair or to reposition the patient without sounding the alarm, press the RESET button. The unit will go into standby mode.

- (a) If the patient's weight is removed from the Sensormat, the unit will immediately begin to monitor again when the weight returns.
- (b) If weight hasn't been removed, press the RESET button a second time to re-start monitoring.
- (c) If the patient has not been removed within approximately one minute, the Chair-Check II will automatically resume monitoring of the patient. The monitoring mode will be indicated by a beep and the red flashing light.

Do not position either component of the system in such a manner that causes one to pull against the other, causing stress to the Sensormat cord or its connecting plug.

7. Nursing personnel should test the system daily for proper functioning. If the patient weighs 100 pounds or more, the nurse may test the system using just hand pressure on the Sensormat. If the patient weighs less than 100 pounds, system function should be checked using the actual patient's presence on the Sensormat. If the system does not perform properly, a different Sensormat may resolve the problem.

8. If the patient gets out of the chair, the Chair-Check II will alarm for approximately ten minutes or until the RESET button is pressed.