

when the Classic-Check Unit is not in 5 second hold, countdown or alarm. One second is strongly recommended if monitoring a patient in a bedside chair or wheelchair, since he/she is poised to get up. 2 or 3 seconds are usually better for bed patients. Shorter delays may increase alarm frequency for restless bed patients, but may be necessary for “fast exit” patients.

**ALARM TONE SETTINGS** (If you do not wish to change the tone setting, skip to **VOLUME SETTINGS**). There are ten available alarm tone settings. Six of them are songs, to provide a sound distinctly different from other equipment in the vicinity. The programmed songs have been selected for their non-threatening nature, while maintaining alarm effectiveness. The tone selected will be the alarm in both **DAY** and **NITE** modes, although volumes may be set separately.

**1. To hear the current alarm tone setting,**

- a. Press and hold **RESET**.
- b. Press and release **ALARM DELAY**.
- c. Release **RESET**.

You will hear the current tone setting. Silence by pressing **RESET**. Note that the volume may be wrong at this point, but it is best to set volume after the tone has been selected. Alarm tones are always played at lowest volume during alarm tone selection regardless of Day/Nite or volume settings.

- d. If this is the tone desired, press **RESET** once more, skip the rest of this section, and proceed to **VOLUME SETTINGS**.

**2. To change the tone setting,**

- a. Press and hold **RESET**.
- b. Press and hold **ALARM DELAY**.
- c. Release **RESET**.
- d. Continue holding **ALARM DELAY** and press/release **RESET** until desired alarm tone is reached.

Each press/release of **RESET** advances the tone one step. If at the end of the list, the sequence will start over at 1.

- e. For a visual indication of what setting you’re on, observe the 1 green and 3 red LEDs in a row.

<u>Setting</u>	<u>○=off ●=on</u>	<u>Tone</u>
1	○○○●	Low pitch warble
2	○○●○	High pitch warble
3	○○●●	Triple tone
4	○●○○	Old McDonald Had a Farm
5	○●○●	Mary Had a Little Lamb
6	○●●○	The Farmer in the Dell
7	○●●●	She’ll Be Comin’ ‘Round the Mountain
8	●○○○	Tom, Tom, the Piper’s Son
9	●○○●	Folk Song
10	●○●○	DoReMiFaSoLaTiDo

- f. When reaching the desired alarm tone, release all buttons. Press/release **RESET** once to silence and lock the tone.