



# QUICK START PROCEDURES

## HOW TO GET STARTED WITH NO SETTING CHANGES, FOR THE EXPERIENCED BED-CHECK CAREGIVER.

**Do use this section** if you already understand the basic operational procedures, and the present settings do not need to be changed.

**Do not use this section** if you are unfamiliar with this equipment or wish to change settings. Please refer to other sections of this User Guide. See Table of Contents.

### CURRENT SETTINGS:

**If brand new**, the factory set delay is 3 seconds, Day mode, “Low pitch warble” alarm tone, maximum Day Volume and minimum Nite Volume.

**If the unit has already been in service**, the settings will be as last selected, whether or not power has been interrupted since then.

Action step is in normal type. *System indications are in Italics.*

### Set up:

1. Install and connect the Sensormat®.
2. Plug in the power cord. *Unit will beep twice with solid yellow (IdleHOLD).*
3. Plug in the nurse call system cord, if used, with an adapter if needed.

### Monitoring patient/resident:

4. Place the patient on the mat. *Flashing yellow for 5 seconds (or push RESET to bypass and monitor immediately), then blinking green.*
5. **CAUTION: Assure the green MONitoring light is blinking before leaving the patient. The system will not alarm while in any HOLD mode (yellow light blinking or on).**

### Repositioning or removing patient/resident:

6. Push **RESET (middle button)**. *Flashing yellow light (temporary HOLD) for 25 seconds. Classic-Check then automatically goes back to MONitor mode (blinking green) if weight is sensed on the Sensormat, or idle HOLD (steady yellow) if no weight is sensed*
7. Any time weight is on the Sensormat, start immediate monitoring by pressing RESET.

### Shutting off alarm:

8. Press RESET. *Flashing yellow (temporary HOLD) for 25 seconds, then steady yellow (idle HOLD) if no weight is on the mat, or blinking green light (MONitor) if weight is on the mat.*

### Day/night Mode change (either way)

9. Press DAY/NITE button once. *DAY mode, no light in splash shield. NITE mode, blue in splash shield.*

**WARNING:** During any HOLD mode (any yellow light), the system will not alarm. It will either remain on HOLD if no weight is on the mat, or begin to monitor after the time intervals cited above if weight is back on the mat. Any time patient is on the Sensormat, you may bypass the wait and begin monitoring immediately by pressing RESET. Always verify the green light (MONitor mode) is blinking before leaving the patient.

ALWAYS PHYSICALLY INSPECT THE STATUS OF THE CONTROL UNIT BY VIEWING THE ILLUMINATED INDICATORS ON THE FACE OF THE CONTROL UNIT BEFORE LEAVING THE PATIENTS ROOM. IF THE CONTROL UNIT IS IN EITHER HOLD MODE, PRESS THE RESET BUTTON TO IMMEDIATELY BEGIN MONITORING AND ENSURE THE GREEN MONITOR LIGHT IS FLASHING. There is a possibility of the patient exiting the bed or chair during either HOLD period.