

POSSIBLE INTERNAL SET-UP INSTRUCTIONS


(only if connecting to a nurse call system)

1. Determine if your nurse call system is a normally closed or normally open system (consult your BioMed). The Classic-Check Unit is factory set for a normally open system unless otherwise specified.
2. If your system is normally open, or if your Classic-Check Unit was ordered to be specifically compatible with your system, skip the rest of this section.
Go to HOW TO USE THE SYSTEM, page 5.

WARNING: Danger of electrical shock. Unplug unit before servicing.

NOTE: The following steps (3 Thru 8) should be accomplished only by your BioMed Department or Bed-Check Corporation.

3. With the Classic-Check Unit unplugged, remove the 4 screws from the back of the case.
4. Remove splash guard and the back cover.
5. At JP1 remove jumper.
6. Re-install the jumper on pins 2 & 3 for normally closed (1 & 2 is for normally open).
7. If RTV has been damaged during disassembly, reapply before reassembling.
Reassemble the cover (fits only one way).
8. Reinstall the 4 screws. The round-headed screws fit the splash shield, and the countersunk screws secure the bottom of the case. Do not over-tighten.

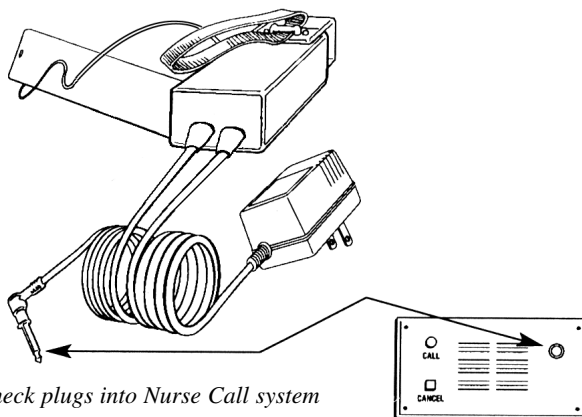

Classic-Check plugs into Nurse Call system