



PROGRAMMING PROCEDURE

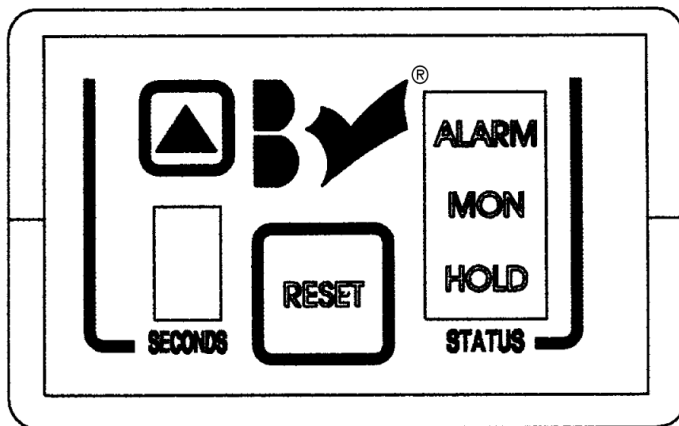


Front panel of the Bed-Check Model Vr Control Unit.

Apply power to the Control Unit. It is not necessary to connect the nurse call cord or to connect a Sensormat.

1. Use the up arrow button  on the face of the Control Unit to set the digital display to nine seconds.
2. Unplug the power supply from the Control Unit for several seconds.
3. While pressing the RESET button on the front panel, apply power to the Control Unit. Release the RESET button after all indicator lights on the right side of the front panel come on, and the unit beeps.

NOTE: All three indicator lights (ALARM, MONitoring, HOLD) should be lit at this time. If not, start over with step one.

4. You are ready to program the sound option. If you do not wish to change the sound, press RESET to skip this step and go to Step 5.
To make a change: The numeric display on the left side of the front panel shows the current sound selection. Match the numeric display with the list below. Use the up arrow buttons  to change the number displayed and choose the desired sound from the list below. Once the appropriate number is displayed, press the RESET button, a beep will follow.