


Sounds:

0. Fast Beep
1. Medium Beep
2. Slow Beep
3. Ring
4. Warble
5. Triple Tone
6. Music


The ALARM indicator should go out leaving the MONitoring and HOLD indicators lit.

5. If you do not want to change the relay action, press RESET to skip this step and go to Step 6. To set the appropriate relay action, use the up arrow button  on the front panel, select the number for the desired relay action from the list below. Press RESET when the correct number is displayed, a beep will follow.

Relay Option:

0. Continuous closure
1. Repeated pulses
2. One Shot
3. No pull-out protection (do not use/or no nurse call system)

The MONitoring indicator should go out and the ALARM and HOLD indicators should remain lit.

6. If you do not want to change the HOLD time, press RESET to skip this step and go to Step 7. Use the up arrow  select the desired HOLD time. The HOLD time is adjustable from 1 to 9 seconds. Programming through a PC provides from 1 to 20 seconds HOLD time. This is the time in seconds that will elapse from the 20 seconds HOLD time. This is the time in seconds that will elapse from the unit will automatically begin MONitoring. After selecting the desired HOLD time, press RESET, a beep will follow.

The HOLD indicator should go out and the ALARM and MONitoring indicators should remain lit.