



CARE AND MAINTENANCE

If maintenance or opening of the Control Unit is required, it should be performed in a static-controlled (free) environment by qualified personnel.

The Control Unit and the Sensormat may be cleaned with a damp cloth or sponge using mild disinfectants. Never use alcohol, acidic or harsh petroleum based cleaners. If absolutely essential, the Control Unit may be gas sterilized up to 120° F.

CAUTION

The Bed-Check Control Unit may not be effective with another manufacturer's sensing device, the use of which may cause damage to the Bed-Check Control Unit and will void the Bed-Check warranty.

Danger of electrical shock. Never immerse the power supply or Control Unit in any liquid.

INTERFACING WITH THE NURSE CALL SYSTEM

The Bed-Check Model Vr can be set for a normally open or normally closed nurse call system. The unit is factory preset to function with a normally open system. Please refer to the program manual.

The Control Unit uses a standard, two conductor, 1/4 inch phone plug, the most common type of nurse call connection. If your nurse call system does not provide a compatible input, contact Bed-Check Corporation or your biomedical department to obtain the appropriate adapter.

If your facility does not have a biomedical department, please contact Bed-Check Corporation at the phone number listed on the back of this user guide. If requesting information on interfacing the Control Unit to a nurse call system, please have the following information available:

- The brand of your nurse call system.
- A description of the system's call cord or pillow speaker, including the type of plug and number of pins in the plug.
- Whether your nurse call system is normally open or normally closed.