

How the Basic-Check System Can Work For You

The Basic-Check system consists of a control unit and a disposable pressure sensitive Sensormat. When a patient's weight is removed from the Sensormat, the control unit activates an alarm.

How to Use the Basic-Check System

Danger: This monitor may not be compatible with mats manufactured by companies other than Bed-Check Corp. Use of incompatible mats may result in patient injury or death.

1. Plug the Sensormat connector into the top of the control unit.
2. Place the Sensormat so that it will be beneath the patient's buttocks.
3. The control unit will automatically arm itself when weight is placed on the Sensormat.
4. If the patient lifts his weight off the Sensormat, the Basic-Check will immediately alarm.
5. To silence the alarm, simply press the red button on top of the control unit.
6. Test the system daily for effective operation.
7. System may not be effective for patients less