

# Basic-Check® User Guide



1-800-523-7956

www.bedcheck.com

## How the Basic-Check System Can Work For You

The Basic-Check system consists of a control unit and a disposable pressure sensitive Sensormat. When a patient's weight is removed from the Sensormat, the control unit activates an alarm.

## How to Use the Basic-Check System

**Danger: This monitor may not be compatible with mats manufactured by companies other than Bed-Check Corp. Use of incompatible mats may result in patient injury or death.**

1. Plug the Sensormat connector into the top of the control unit.
2. Place the Sensormat so that it will be beneath the patient's buttocks.
3. The control unit will automatically arm itself when weight is placed on the Sensormat.
4. If the patient lifts his weight off the Sensormat, the Basic-Check will immediately alarm.
5. To silence the alarm, simply press the red button on top of the control unit.
6. Test the system daily for effective operation.
7. System may not be effective for patients less

than 100 lbs. Assure proper operation before putting into service.

8. Do not position either the unit or the Sensormat in such a manner that causes one to pull on the other, causing stress to the connector wire or the plug.

## Care and Maintenance

The control unit can be wiped with a damp cloth or sponge using mild detergents or Sanitizing solutions. Never use alcohol or strong chemical cleaners. NEVER IMMERSE THE CONTROL UNIT IN ANY LIQUID. AVOID PLACING THE CONTROL UNIT IN ANY AREA WHERE IT MIGHT COME IN CONTACT WITH BODY FLUIDS. DAMAGE BY FLUIDS VOIDS THE WARRANTY.

- ✓ Internal maintenance should only be performed by qualified personnel.
- ✓ Bending or folding the Sensormat may cause loss of effective operation.

## Battery

**NOTE: The Basic-Check control unit includes an alkaline battery. A low battery is indicated by a low constant beeping, even while the patient is on the Sensormat. DO NOT ATTEMPT TO RECHARGE THE BATTERY.**

## WARRANTY

Basic-Check Control Unit and Sensormat.

1. Warrantor: This limited Warranty is given by Bed-Check Corp., 307 E. Brady Tulsa, OK. 74120, (800)523-7956

2. Duration: Basic-Check Control Unit: begins on date delivered to purchaser and continues for a period of 2 years.

Sensormat: begins on date delivered to purchaser and continues for the lesser of 1 year or as specified on the Sensormat label from the installation date. To whom this warranty is given: The limited warranty is given to the original purchaser of Bed-Check products only. What is covered: Defects in material and workmanship. Warrantor makes no other warranties express or implied, including merchantability or fitness of product for a particular purpose. What is not covered:

- A) Any incidental, indirect or consequential loss, damage, or expense that may result from any defect, failure, or malfunction.
- B) Any incidental, indirect or consequential loss, damage, or expense that may result from use of product with another manufacturer's product. NOTE: Some states do not allow the exclusion or limitation of incidental or consequential damages.
- C) Any failure that may result from an accident, purchaser's abuse, neglect or failure to

operate the product in accordance with the instructions provided herein. D) Any product which has the serial number altered, defaced or removed. E) Any product which has been altered or modified without the written consent of Bed-Check Corp. Responsibilities of warrantor: a) In the event of a defect, malfunction, or other failure not caused by any misuse or damage the warrantor will remedy the failure or defect without charge within a reasonable period of time. The remedy will consist of repair, replacement or refund at the warrantor's option. If the product is no longer available, warrantor will supply a comparable product or refund the purchase price, at his option. Responsibilities of the purchaser: A) Disinfect the product, if necessary, so that it is free of infectious matter. B) Ship the Sensormat unfolded. C) Use the product with reasonable care. This limited warranty gives you specific legal rights. You may also have other rights which vary from state to state.

100233 REV C